

# DermaCLIP<sup>®</sup> EXTREME<sup>™</sup>

## QUICK REFERENCE GUIDE

SEE IFU FOR  
FULL INSTRUCTIONS

- FOR FINAL OR TEMPORARY CLOSURE OF WOUNDS, CUTS AND INCISIONS.
- **SEEK MEDICAL ATTENTION FOR DEEP, LARGE, INFECTED, BITE, PUNCTURE, HEAVILY BLEEDING OR CRUSH WOUNDS.**
- **DO NOT USE IF ALLERGIC TO ADHESIVES.**
- **DO NOT USE IF STERILE PACKAGE IS OPEN OR DAMAGED.**
- **NOT SUITABLE FOR FRAGILE, DAMAGED, OR BURNED SKIN.**

BEFORE APPLYING THE DEVICE: FLUSH THE WOUND, CONTROL THE BLEEDING BY APPLYING PRESSURE, CLEAN THOROUGHLY WITH ALCOHOL SWAB, REMOVING DIRT & DEBRIS, AND DRY AREA THOROUGHLY.

TO APPLY THE DEVICE:




### PLACE

PLACE DEVICE EDGES ON EACH SIDE OF THE WOUND, AS CLOSE TO WOUND EDGE AS POSSIBLE.

PLACE AS MANY DEVICES AS NEEDED, 1 MM TO 4 MM APART.

**PRESS  
FIRMLY  
ONTO SKIN**

ADHESIVE IS PRESSURE SENSITIVE. MASSAGE PADS **FIRMLY** INTO SKIN TO ENSURE ADHESIVE ACTIVATION.

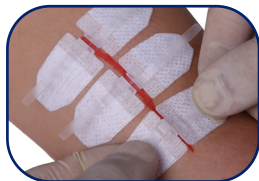

### PULL TILL CLICK

PULL TABS STRAIGHT OUT (NOT UP) UNTIL CLICK IS HEARD AND FELT TO INDICATE LOCKING.



DEVICE  
SLIDES OPEN

**CHECK DEVICE LOCKING**  
PUSH DOWN ON CENTER  
OF DEVICE TO ENSURE  
DEVICE IS FULLY LOCKED.




REMAINS LOCKED,  
RIGID, & EVERTED

#### PATIENT CARE INSTRUCTIONS:

- DO NOT USE PETROLEUM BASED OINTMENTS SUCH AS BACITRACIN OR NEOSPORIN®.
- KEEP WOUND CLEAN AND DRY. CHECK WOUND SITE DAILY.
- IF WOUND IS RED, HOT, OR DRAINING, SEEK MEDICAL ATTENTION.
- DO NOT RUN WATER DIRECTLY ON THE DEVICE.  
IF WET, BLOT DRY; NEVER RUB.

#### REMOVAL INSTRUCTIONS:

- ONLY REMOVE WITH PERMISSION OF MEDICAL PROFESSIONAL.
- LIGHT SKIN OIL EASES REMOVAL.
- PEEL EACH ADHESIVE PAD TOWARDS CENTER OF THE CLOSED WOUND.
- **DO NOT PULL IN ONE DIRECTION, AS THIS PULLS AGAINST THE CLOSURE OF THE HEALING WOUND ON ONE SIDE.**
- THEN PEEL OFF FROM FRONT TO BACK OF THE WOUND, DO NOT PULL.

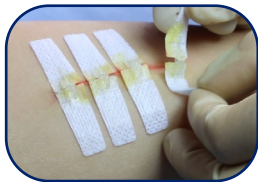

**FOR FULL INSTRUCTIONS FOR USE  
(IFU) PLEASE USE THE QR CODE.**



**DermaClip®**  
www.DermaClip.com  
info@DermaClip.com  
+1 (713) 682-3185

**Disclaimer:** For detailed instructions on using the device as well as all relevant warnings and contraindications, please visit [DermaClip.com](http://DermaClip.com).

Doc No.: DCUS-IFU-EXT4X6 Rev A