

- FOR MINOR CUTS & LACERATIONS ONLY.
- **SEEK MEDICAL ATTENTION FOR DEEP, LARGE, INFECTED, BITE, PUNCTURE, HEAVILY BLEEDING OR CRUSH WOUNDS.**
- **DO NOT USE IF ALLERGIC TO ADHESIVES.**
- **DO NOT USE IF STERILE PACKAGE IS OPEN OR DAMAGED.**
- **SPORT & EXTREME ADHESIVE IS NOT SUITABLE FOR FRAGILE OR DAMAGED SKIN.**

TO APPLY THE DEVICE:



CLEAN

FLUSH WOUND THOROUGHLY, CONTROL BLEEDING, CLEAN THOROUGHLY WITH ALCOHOL SWAB, REMOVING DIRT & DEBRIS. DRY AREA.



PLACE

PLACE DEVICE EDGES ON EACH SIDE OF THE WOUND. MASSAGE PADS FIRMLY INTO SKIN, AS ADHESIVE IS PRESSURE SENSITIVE.



PRESS

PRESS EDGES OF DEVICE TOGETHER UNTIL DEVICE EDGES MAKE CONTACT.



POP-UP

POP-UP THE CENTER OF DEVICE TO CLOSE AND LOCK.

TO APPLY WOUND DRESSING:

- PEEL OPAQUE PAPER OFF SIDE WITH GAUZE PAD.
- APPLY WITH GAUZE PAD OVER THE LENGTH OF THE WOUND; MAKE SURE DRESSING ADHESIVE DOES NOT TOUCH DEVICE.
- PEEL CLEAR BACKING OFF TOP OF PAD AND SMOOTH DRESSING ONTO SKIN.

PATIENT CARE INSTRUCTIONS:

- DO NOT USE PETROLEUM BASED OINTMENTS SUCH AS BACITRACIN OR NEOSPORIN®.
- KEEP WOUND CLEAN AND DRY. CHECK WOUND SITE DAILY.
- IF WOUND IS RED, HOT, OR DRAINING, SEEK MEDICAL ATTENTION.
- DO NOT RUN WATER DIRECTLY ON THE DEVICE. IF WET, BLOT DRY; NEVER RUB.

REMOVAL INSTRUCTIONS:

- LIGHT SKIN OILS HELP EASE REMOVAL.
- PEEL EACH ADHESIVE PAD TOWARDS THE CLOSED WOUND.
- PEEL OFF, DO NOT PULL.



PEEL TO CENTER

PEEL EACH PAD TOWARDS
CENTER OF WOUND SITE



PEEL OFF

PEEL OFF,
DO NOT PULL

**FOR FULL INSTRUCTIONS FOR USE
(IFU) PLEASE USE THE QR CODE.**



DermaQuik[™]
www.DermaQuik.com
info@DermaQuik.com
+1 (713) 682-3185

Disclaimer: For detailed instructions on using the device as well as all relevant warnings and contraindications, please visit DermaQuik.com.

Doc No.: DQOC-IFU-OTC4X6 Rev B