



MAKING CLOSURE A SIMPLE PROCESS

QUICK & EASY

- ✓ No Needles or Local Anesthetic
- ✓ Wound Eversion on Closure
- ✓ Works on Most Skin Types
- ✓ Curved Wounds & Wider Gaps

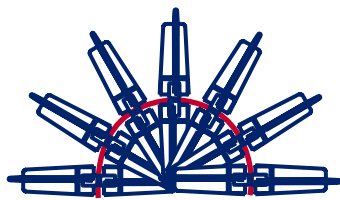


Curved Wounds

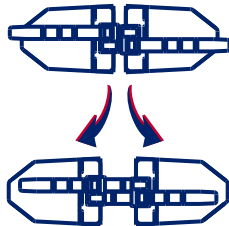


Separate Pads for Wider Gaps



MULTIPLE SIZES & OPTIONS

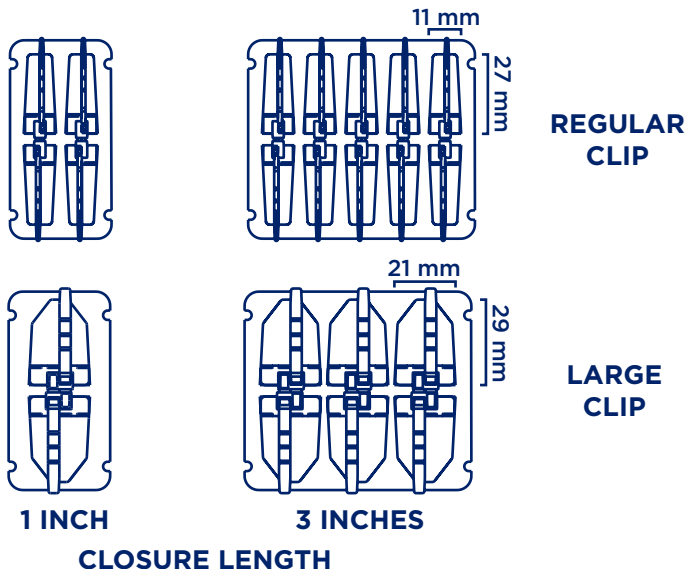

Place on wound edges & smooth firmly

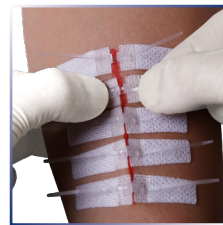


Grasp tabs at center, pull tabs **OUT** (NOT UP) until "Click" is heard & felt

Clip excess tabs after device is locked

CHECKING IF DEVICE IS LOCKED

Push down on center.
If it separates, **NOT** locked.

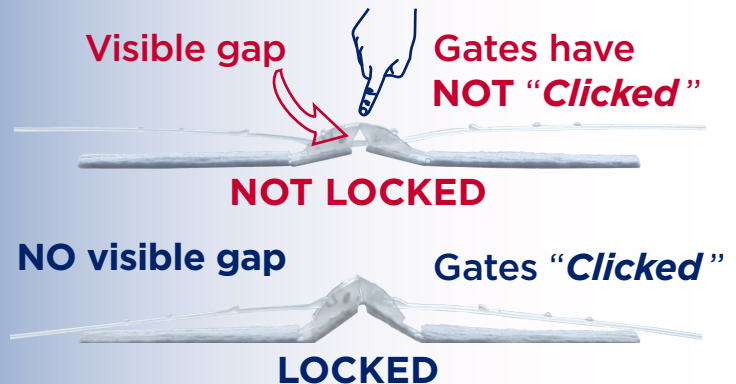

CARE INSTRUCTIONS

- ✓ Same as Sutures
- ✗ Do **NOT** use on or under devices:
 - ✗ **OINTMENTS**
(BACITRACIN, NEOSPORIN®, ETC.)
 - ✗ **CREAMS / LOTIONS**
- ✗ Adhesive of dressing should **NOT** touch device pads