## **GENERAL INSTRUCTIONS**

Normally, after a wound or incision is closed with DermaCLIP®, nothing needs to be done until the devices are removed. The following care guidelines will ensure the best performance of the device.

- **DO NOT** use any cream, lotion or antibiotic ointments while the devices are in place. Use of these solutions could cause device failure.
- Keep the devices clean and as dry as possible, especially for the first 24 hours.
- Take care not to stress or flex the skin around the area of the devices.
- After 24 hours, or when allowed to begin showering by your medical professional, it is possible to gently wash the area around the devices with soap and water. **DO NOT** wash or rub the devices directly.
- When showering or rinsing, covering the DermaCLIP® devices will prolong their life.
- **DO NOT** soak devices in water. **DO NOT** swim or use a hot tub or bathtub while the devices are in place.
- When drying the devices, pat them with a clean towel. **DO NOT** rub them.
- If placing a gauze or bandage over the devices, **DO NOT** apply any tape, sticky dressing, or alcohol to the DermaCLIP® devices. **MAKE SURE** none of the adhesive of the tape or sticky dressing comes into contact with the DermaCLIP® devices.
- If a device splits open or falls off before the scheduled time for removal, contact your medical professional.
- Follow the wound care instructions given to you by your medical professional.

## **REMOVAL INSTRUCTIONS**

At the direction of the medical professional, the DermaCLIP® devices may be removed by the patient. If you, the patient, are instructed to remove them, follow the guidance below.

- DermaCLIP® devices should normally be left on the wound at least 7 days or as directed by the medical professional. **DO NOT** remove them before specified by the provider.
- Before removal, apply a light oil (i.e., baby oil or lotion) to facilitate removal and protect the skin.
- Devices should be **GENTLY PEELED** from the outside of the device towards the middle of the wound on each side. Carefully removing from both sides protects the wound itself.
- **DO NOT** remove the device by pulling it from the skin or from the middle, as damage to the healed wound and reopening may result. **“PEEL; DON’T PULL!”**

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