

Quick Application Guide

To apply the DermaClip® device, simply:



1

After cleansing & patting wound dry, pick the device off the paper.



2

Place it on the dry skin, smoothing pads to ensure a good bond.



3

Pull the tabs "**Down & Outward**" until device "*clicks*" locked.



4

Clip the excess tabs and cover the wound with a dressing.


Do not use with petroleum-based ointments, creams or other petroleum based products (e.g., Bacitracin).

If body hair is present in wound area, ***depilate area before applying.***

For more information, visit www.DermaClip.com/instructions

Visual & Manual Check to Confirm Device is Locked Before Cutting Tabs:

Push Down to Check

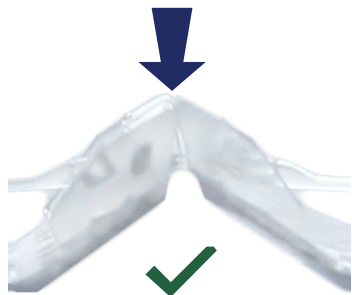


Device Remains Rigid
Locked & Everted

PROPERLY LOCKED

- No gap; faces in full contact
- Full eversion has occurred
- Gates have snapped closed
- Adhesive pads nearly touch
- ***Remains rigid & everted when manually tested (pushed)***

Push Down to Check

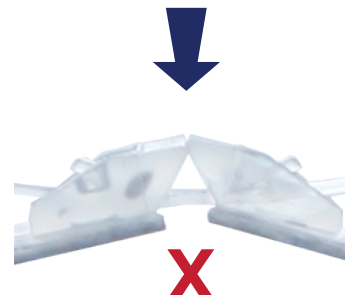

Device Slides
Opened

NOT LOCKED

- Gates have **not** snapped closed
- Gap between faces; not in full contact
- ***Slides apart when manually tested (pushed)***