

If your medical provider instructed you to remove the DermaCLIP® devices at home, please follow these instructions on the prescribed day for removal. **DO NOT** remove the devices before the day specified by your provider.

If you have any questions or concerns, contact your medical provider directly.

REMOVAL INSTRUCTIONS

- **APPLY A LIGHT OIL** (i.e., baby oil or lotion) to breakdown the adhesive to help release the devices from the skin. Allow it to soak into the pads.
- **GENTLY PEEL** each side of the device towards the center (*Pictures 1 & 4*). Carefully removing from both sides protects the wound.
- **DO NOT PULL ACROSS THE INCISION LINE** as this will pull against the closure that was just completed and reopening may result.
- Once each pad has been peeled towards the incision line, **GENTLY LIFT THE CENTER PORTION THAT COVERS THE INCISION.**
- **ALWAYS PEEL SLOWLY; DON'T PULL!**

